

User Guide

Medit Link



■ Medit Link 界面介紹 ■

台灣 紐白特

官網連結：[Medit Link註冊](#) 方法

Medit Link

+註冊選擇
菜單選項



請注意，菜單項因您擁有的帳戶而異。

Medit Link

+ Dashboard
綜合狀態數據



MEDIT Link 3.1.0 neobiotech104@gmail.com

雲端上傳&下載 網路連接狀態 通知中心 使用者ID 雲端儲存容量 統計數據

儀表板 使用口掃機資訊

Scanner

i500 (AA1805100151)

Warranty Inquiry Expired
Usage Time 2d 4h 55m
Calibration Period 14 Days
Elapsed Days 192 Days

Cloud Storage

141.24 MB / 1 GB

Statistics

| | |
|------------------------------|-----------|
| Total Cases | 62 |
| Scanned Cases | 43 |
| Working Time | 373:01:54 |
| Total Orders | 25 |
| Completed Orders | 4 |
| Canceled Orders | 0 |
| Rejected Orders | 0 |
| Total Purchased Amount (USD) | 0.00 |

Order Status

| | |
|-----------|----|
| Pending | 1 |
| Accepted | 20 |
| Shipped | 0 |
| Completed | 4 |

Graph

2023-05-16 ~ 2023-06-15

| Total Cases | Scanned Cases | Working Time | Total Orders | Total Purchased Amount |
|-------------|---------------|--------------|--------------|------------------------|
| 0 | 0 | 00:00:00 | 0 | 0 |

案件總數 掃描件 工作時間 總訂單 採購總額

Tooth Type

牙齒類型

2023-05-16 ~ 2023-06-15

No search results were found.

訂閱Medit

Show data for Graph and Tooth Type up to 2023-06-15. ?

Subscribe Medit Insights

在病例區(Case Box) 中，您可以創建、查找和管理病例。對於每個病例，會簡要顯示患者信息、修復類型信息和病例狀態。

Medit Link

+Case Box 工作文件

- [A] 搜索病例
- [E] 排序順序
- [B] 按狀態分類病例
- [F] 查看方式
- [C]分組案例
- [G] 病例列表
- [D] 創建新的病例

The screenshot shows the 'Case Box' interface in Medit Link. The interface includes a sidebar with navigation options (Dash Board, Case Box, Order Box, Case Talk, App Box, Trash Box), a top navigation bar with 'MEDIT Link' and user information, and a main content area. The main content area features a filter bar with status buttons (All, Form Only, Need Processing, Scan Completed, CAD, Ordered, Case Completed), a search bar, and a table of cases. Annotations A-G point to specific UI elements: A (Search), B (Filter by status), C (Grouping), D (New Case button), E (Sort), F (Case Box icon), and G (Case list row).

| Status | Case Name | Patient Name | Birth Date | Form Information | Date Modified ↓ | Date Scanned | Date Ordered | Lab Name |
|-----------------|-----------------------------|-------------------------|------------|------------------|---------------------|---------------------|---------------------|------------|
| Need Processing | APPS - Temporary | 77777 | - | - | 2021-10-14 上午 3:16 | 2021-08-24 上午 6:20 | | |
| Need Processing | preop / impression scan | 77777 | - | - | 2021-10-14 上午 3:02 | 2021-08-24 上午 6:20 | | |
| Scan Completed | i700 model practice | 77777 | - | - | 2021-10-14 上午 3:01 | 2021-09-07 上午 4:13 | | |
| CAD | APPS - Crown fit | 77777 | - | - | 2021-10-14 上午 2:17 | | | |
| CAD | Crown Fit | | - | - | 2021-10-14 上午 2:16 | | | |
| Processing | 00's Case | 00 | - | - | 2021-10-14 上午 12:23 | 2021-10-14 上午 12:19 | | |
| Form Only | TEST's Case | TEST | - | - | 2021-10-13 上午 6:04 | | | |
| Form Only | TEST's Case | TEST | - | - | 2021-10-13 上午 4:23 | 2021-10-13 上午 4:04 | | |
| Form Only | Abutment's Case | Abutment | - | - | 2021-10-13 上午 4:12 | | | |
| Scan Completed | Ortho Simulation - ext, ... | 77777 | - | - | 2021-10-13 上午 4:09 | 2020-02-07 上午 3:10 | | |
| Scan Completed | TEST's Case | TEST | - | - | 2021-10-13 上午 3:55 | 2021-09-29 上午 1:13 | | |
| Scan Completed | WH Choi - complete - cl... | Wonhoonchoi | - | - | 2021-10-13 上午 3:26 | 2019-09-27 上午 4:56 | | |
| Scan Completed | trst's Case | demo test | - | - | 2021-10-13 上午 2:50 | 2021-10-12 上午 12:50 | | |
| Ordered | #26 Cr' | Hyohyang Lee(ID 920915) | 1992-09-15 | - | 2021-10-13 上午 2:34 | 2021-04-27 上午 11:06 | 2021-10-13 上午 11:25 | medit lab1 |

D 創建新的病例

Medit Link

+Case Box

工作文件

The screenshot shows the Medit Edu software interface. At the top, there is a window title bar with 'Medit_Edu' and standard OS window controls. Below the title bar, there is a navigation bar with a 'New Case' button (labeled D) and a 'None' filter (labeled C). Below the navigation bar, there is a search bar with a date range '2018-01-01 - 2021-10-14' and a search type 'All' (labeled A). Below the search bar, there is a table with columns 'Date Modified', 'Date Scanned', 'Date Ordered', and 'Lab Name'. The table contains several rows of data.

| Date Modified ↓ | Date Scanned | Date Ordered | Lab Name |
|--------------------|--------------------|--------------|----------|
| 2021-10-14 오후 3:16 | 2021-08-24 오후 6:20 | | |
| 2021-10-14 오후 3:02 | 2021-08-24 오후 6:20 | | |
| 2021-10-14 오후 3:01 | 2021-09-07 오후 4:13 | | |
| 2021-10-14 오후 2:17 | | | |

New Case Information

1 Patient Name*

3



2 Case Name*

1 患者名字

2 病例名稱

3 顯示患者列表

Medit Link

+Case Box

工作文件



The screenshot shows the Medit Link software interface. The top navigation bar includes '工作文件' (Work File), '工作檔名' (Work File Name), '類型' (Type), '方法' (Method), '材料' (Material), '顏色' (Color), '訂單' (Order), and '掃描' (Scan). The main workspace is divided into 'Form' and 'File Viewer' tabs. The 'Form' tab displays a dental arch diagram with teeth numbered 11-28 and 31-48. Labels include '牙位選擇' (Tooth Selection), '牙齒' (Teeth), and '牙弓' (Arch). The 'File Viewer' tab shows a 'Type' menu with various dental procedures like Inlay, Crown, Implant Crown, etc., and a 'Color' menu. A 'Favorites' section is labeled '喜好項目' (Favorite Items). Below the menu is an 'Order List' table with columns for No., Type, Method, Material, and Shade. The bottom right corner has '保存' (Save) and 'Cancel' buttons.

| No. | Type | Method | Material | Shade | |
|-----|-----------------|--------|----------|-------|----|
| 38 | Custom Abutment | - | Titanium | - | 🗑️ |
| 37 | Custom Abutment | - | Titanium | - | 🗑️ |
| 36 | Custom Abutment | - | Titanium | - | 🗑️ |

功能設定

設定

通知和更新

轉道Medut Link網頁

Medut資訊

Medit Link

+Order Box

訂單資訊



- 設置**
•提供用於整體系統設置的功能。
- 通知和更新**
•顯示程序更新和嚴重警報。
- 幫助中心**
•轉到幫助中心頁面。您可以獲得有關使用Medit Link 的各種信息。

- 我的帳戶**
•將您轉到您當前登錄的帳戶下的Medit Link 主頁。

訂單資訊

案例列表

MEDIT Link 3.1.0 neobiotech104@gmail.com

Order Box

Dashboard Case Box

Order Box Case Talk App Box Trash Box

✓ All | Pending Canceled Rejected Accepted Shipped Case Completed View Options

Total Cases: 25 2018-01-01~2023-06-16 All All / Contains

| Status | Case Name | Patient Name | Order Date | Requested Delivery Date | Lab Name | Order ID |
|----------------|----------------|--------------|---------------------|-------------------------|-----------|----------|
| Accepted | 김중현's Case | 김중현 | 2022-11-28 오후 6:34 | 2022-12-05 오후 5:45 | 주)완선치과기공소 | 10874953 |
| Pending | 221217김영훈의 케이스 | 221217김영훈 | 2022-12-30 오후 5:25 | 2023-01-06 오후 5:15 | 주)완선치과기공소 | 11058785 |
| Accepted | 정봉진의 케이스 | 정봉진 | 2022-11-24 오전 11:14 | 2022-12-01 오후 5:45 | 주)완선치과기공소 | 10861062 |
| Accepted | 최석환의 케이스 | 최석환 | 2022-11-23 오후 5:10 | 2022-11-30 오후 5:45 | 주)완선치과기공소 | 10854210 |
| Accepted | 윤경미의 케이스 | 윤경미 | 2022-11-23 오후 6:02 | 2022-11-30 오후 5:45 | 주)완선치과기공소 | 10854555 |
| Accepted | 이유경의 케이스 | 이유경 | 2022-11-23 오후 2:08 | 2022-11-30 오후 5:45 | 주)완선치과기공소 | 10853759 |
| Accepted | 김에스터's Case | 김에스터 | 2022-11-23 오후 3:56 | 2022-11-30 오후 5:45 | 주)완선치과기공소 | 10853958 |
| Case Completed | 오동섭's Case | 오동섭 | 2022-10-21 오후 5:54 | 2022-10-28 오후 5:45 | 주)완선치과기공소 | 10548933 |
| Case Completed | 박지훈's Case | 박지훈 | 2022-10-26 오전 10:41 | 2022-11-02 오후 5:45 | 주)완선치과기공소 | 10573209 |
| Case Completed | 이세희의 케이스 | 이세희 | 2022-10-28 오후 5:49 | 2022-11-04 오후 5:45 | 주)완선치과기공소 | 10590559 |
| Case Completed | 이재범의 케이스 | 이재범 | 2022-10-31 오후 6:00 | 2022-11-07 오후 5:45 | 주)완선치과기공소 | 10597683 |
| Accepted | 이종향's Case | 이종향 | 2022-11-22 오전 11:45 | 2022-11-29 오후 5:45 | 주)완선치과기공소 | 10843317 |
| Accepted | 오윤경's Case | 오윤경 | 2022-11-18 오후 6:04 | 2022-11-25 오후 5:45 | 주)완선치과기공소 | 10825907 |


官網連結：[Case Talk](#) 使用方法

Medit Link

+Case Talk

共享案例

共享案例

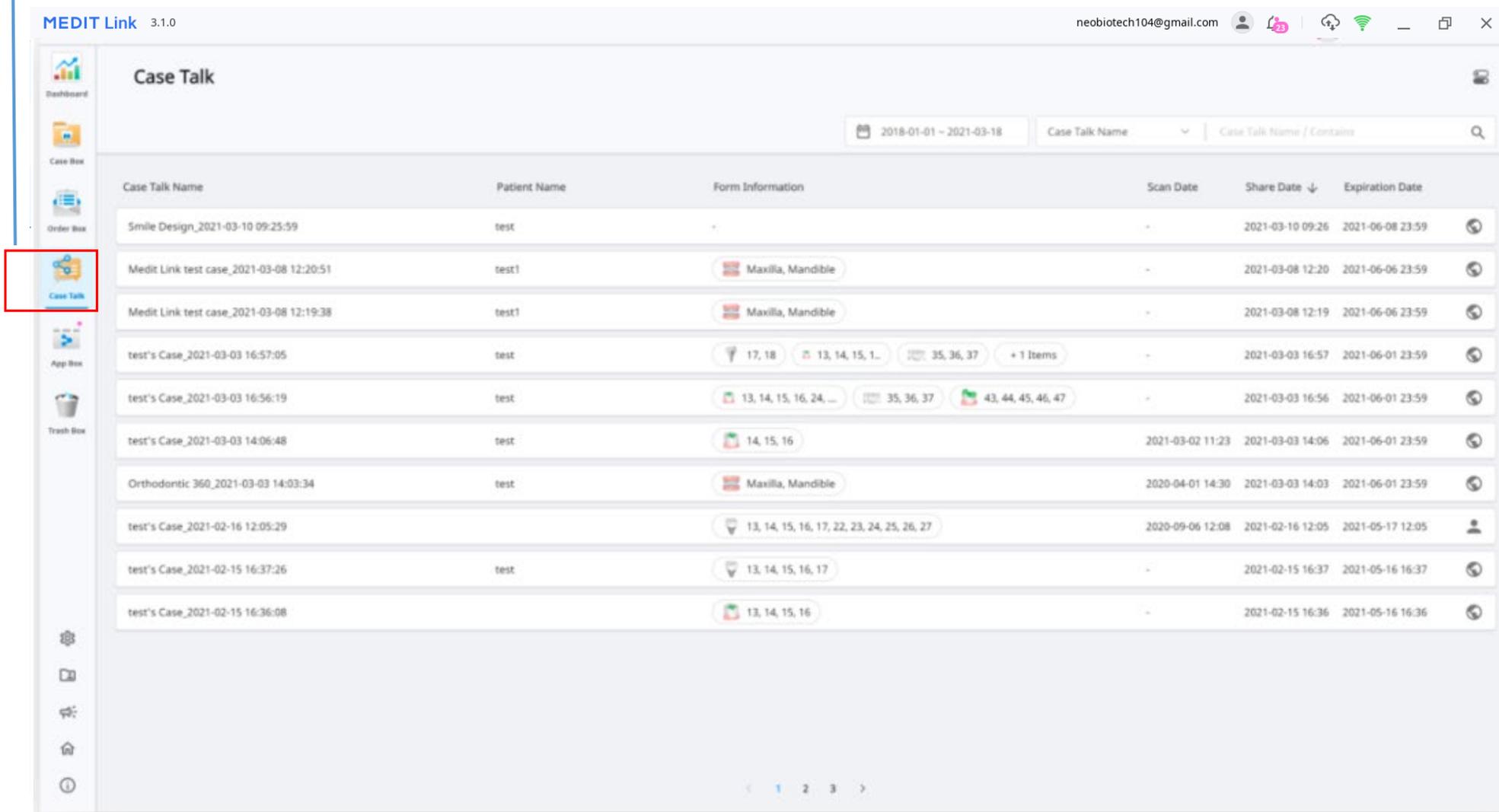

The screenshot shows the MEDIT Link 3.1.0 interface. The top navigation bar includes the user email 'neobiotech104@gmail.com' and various system icons. The main content area is titled 'Case Talk' and features a table of case records. A sidebar on the left contains navigation icons for Dashboard, Case Box, Order Box, Case Talk (highlighted with a red box), App Box, and Trash Box. The table columns are Case Talk Name, Patient Name, Form Information, Scan Date, Share Date, and Expiration Date. The table contains several rows of test cases with various form information and dates.

| Case Talk Name | Patient Name | Form Information | Scan Date | Share Date ↓ | Expiration Date |
|--|--------------|---|------------------|------------------|------------------|
| Smile Design_2021-03-10 09:25:59 | test | - | - | 2021-03-10 09:26 | 2021-06-08 23:59 |
| Medit Link test case_2021-03-08 12:20:51 | test1 | Maxilla, Mandible | - | 2021-03-08 12:20 | 2021-06-06 23:59 |
| Medit Link test case_2021-03-08 12:19:38 | test1 | Maxilla, Mandible | - | 2021-03-08 12:19 | 2021-06-06 23:59 |
| test's Case_2021-03-03 16:57:05 | test | 17, 18; 13, 14, 15, 1...; 35, 36, 37; + 1 Items | - | 2021-03-03 16:57 | 2021-06-01 23:59 |
| test's Case_2021-03-03 16:56:19 | test | 13, 14, 15, 16, 24, ...; 35, 36, 37; 43, 44, 45, 46, 47 | - | 2021-03-03 16:56 | 2021-06-01 23:59 |
| test's Case_2021-03-03 14:06:48 | test | 14, 15, 16 | 2021-03-02 11:23 | 2021-03-03 14:06 | 2021-06-01 23:59 |
| Orthodontic 360_2021-03-03 14:03:34 | test | Maxilla, Mandible | 2020-04-01 14:30 | 2021-03-03 14:03 | 2021-06-01 23:59 |
| test's Case_2021-02-16 12:05:29 | test | 13, 14, 15, 16, 17, 22, 23, 24, 25, 26, 27 | 2020-09-06 12:08 | 2021-02-16 12:05 | 2021-05-17 12:05 |
| test's Case_2021-02-15 16:37:26 | test | 13, 14, 15, 16, 17 | - | 2021-02-15 16:37 | 2021-05-16 16:37 |
| test's Case_2021-02-15 16:36:08 | test | 13, 14, 15, 16 | - | 2021-02-15 16:36 | 2021-05-16 16:36 |

Medit Link

+App Box

軟件下載

- 1 按類別對APP進行分類。
- 2 點擊搜索圖標，利用關鍵字搜索程序。
- 3 APP軟件列表

軟件下載

MEDIT Link 3.1.0

neobiotech104@gmail.com

Search by App Information

All Apps All Categories

| App Name | Category | By | Version | Date | Total Downloads | Install Button |
|--------------------------|---------------|-------|-------------|------------|-----------------|----------------|
| Medit Model Builder | CAD Tool | MEDIT | v 1.3.3.70 | 2023-05-17 | 76,819 | Install |
| Medit Smile Design | Tool | MEDIT | v 1.2.5.75 | 2023-05-17 | 55,795 | Install |
| Medit Ortho Simulation | Tool | MEDIT | v 1.3.1.43 | 2023-05-17 | 49,889 | Install |
| Medit Crown Fit | Tool | MEDIT | v 1.1.2.61 | 2023-05-17 | 26,019 | Install |
| Medit Margin Lines | Tool | MEDIT | v 1.0.1.40 | 2023-05-17 | 12,061 | Install |
| Medit Temporaries | CAD | MEDIT | v 1.1.3.135 | 2023-05-17 | 36,870 | Install |
| Medit Splints | CAD Tool | MEDIT | v 1.0.4.134 | 2023-05-17 | - | Install |
| Medit Occlusion Analyzer | Analysis Tool | MEDIT | v 1.0.2.78 | 2023-05-17 | - | Install |
| Medit Design | Tool | MEDIT | v 2.1.3.95 | 2023-05-11 | - | Install |


Medit Link

+ Trash Box

垃圾桶

垃圾桶

- 1 在單選列表中選擇查詢屬性後，可以通過輸入病例名稱、患者姓名、加工廠名稱或被刪除的病例進行查詢。
- 2 必須選擇至少一個病例，才能激活「還原」和「永久刪除」按鈕。您也可以選擇所有病例。
- 3 顯示從工作區(Work Box)中刪除的病例列表。

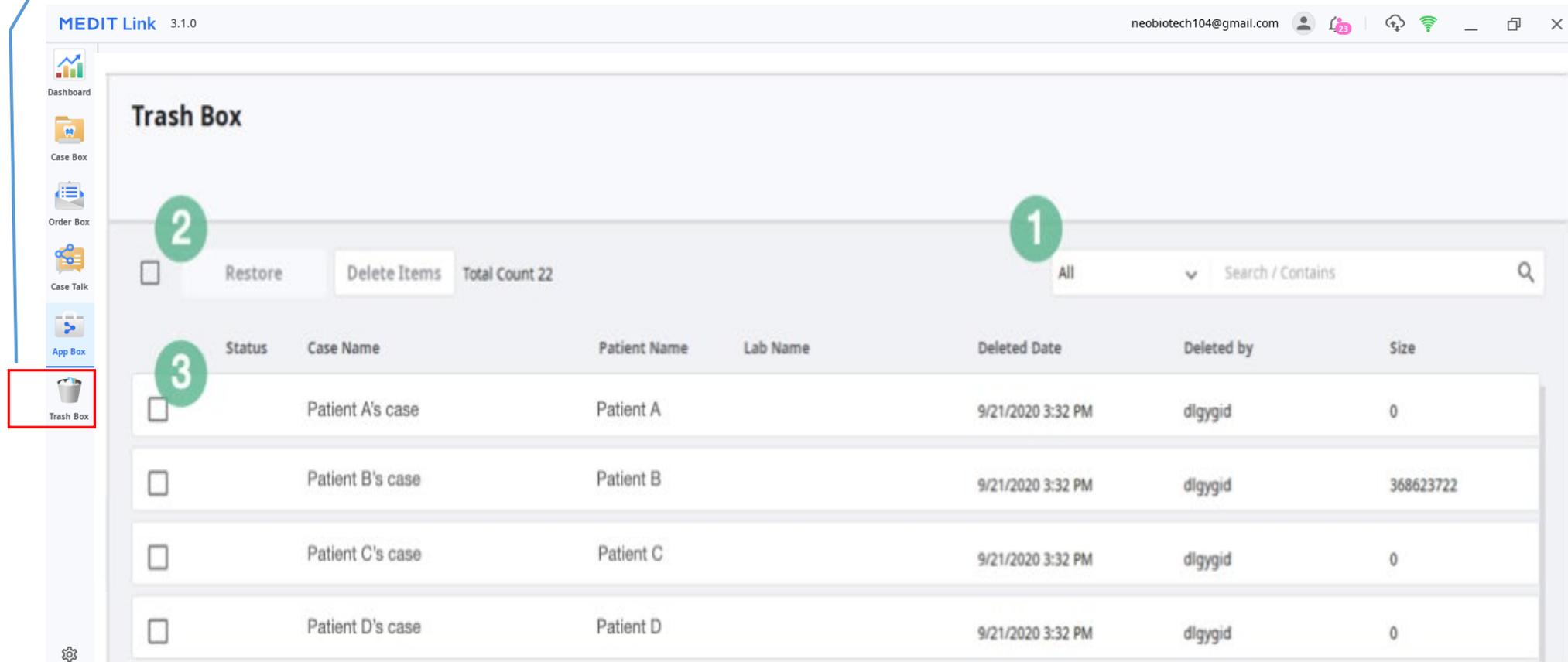

MEDIT Link 3.1.0

neobiotech104@gmail.com

Trash Box

Restore Delete Items Total Count 22

All Search / Contains

| Status | Case Name | Patient Name | Lab Name | Deleted Date | Deleted by | Size |
|--------------------------|------------------|--------------|----------|-------------------|------------|-----------|
| <input type="checkbox"/> | Patient A's case | Patient A | | 9/21/2020 3:32 PM | digygid | 0 |
| <input type="checkbox"/> | Patient B's case | Patient B | | 9/21/2020 3:32 PM | digygid | 368623722 |
| <input type="checkbox"/> | Patient C's case | Patient C | | 9/21/2020 3:32 PM | digygid | 0 |
| <input type="checkbox"/> | Patient D's case | Patient D | | 9/21/2020 3:32 PM | digygid | 0 |

操作方法 ① ~ ⑦ 選擇

最後按 ⑧ 進入i700軟件

Medit Link

i700 App

+Create Project

創建項目

The screenshot shows the MEDIT Link 3.1.0 interface for a case named 'Neobiotech taiwan's Case (Neobiotech taiwan)'. The interface includes a sidebar with navigation options like Dashboard, Case Box, Order Box, Case Talk, App Box, and Trash Box. The main area features a dental arch diagram with teeth numbered 11-28 and 31-48. A red box highlights teeth 36, 37, and 38, with a callout '2' pointing to them. A 'Form' tab is active, showing fields for 'Type', 'Method', 'Material', 'Shade', and 'Favorites'. A 'Scan' button is highlighted with callout '8'. An 'Order List' table is visible at the bottom right, listing items 36, 37, and 38. A 'Save' button is highlighted with callout '7'. Other callouts include '1' for Case Box, '3' for 類型 (Type), '4' for 方法 (Method), '5' for 材料 (Material), and '6' for 顏色 (Shade). Labels '牙齒' (Teeth) and '牙弓' (Arch) are at the bottom, and '掃描' (Scan) is at the top right.

1 Case Box

2 牙齒 (Teeth)

3 類型 (Type)

4 方法 (Method)

5 材料 (Material)

6 顏色 (Shade)

7 保存 (Save)

8 掃描 (Scan)

牙位選擇

牙齒 (Teeth)

牙弓 (Arch)

Order List

| No. | Type | Method | Material | Shade | |
|-----|-----------------|--------|----------|-------|----|
| 38 | Custom Abutment | - | Titanium | - | 🗑️ |
| 37 | Custom Abutment | - | Titanium | - | 🗑️ |
| 36 | Custom Abutment | - | Titanium | - | 🗑️ |

Medit i700



- Medit Scan for Clinics 用戶界面是為使用觸控屏的電腦的用戶提供了觸控功能。
- 標題欄可能看起來不同，並且圖像可以根據macOS 上的屏幕分辨率顯示不同的大小。

+MEDIT Scan for Clinics

標題欄

MEDIT Scan for Clinics 1.10.0

Neobiotech taiwan's Case - [The license will be deactivated within 8 day(s). Please connect the device where the internet connection is available.]

Simple UI

Trimming

Tools

Form Information

Scan Light Source
Utilize white light when scanning bleeding areas in the mouth.

Simple UI 簡單的用戶界面

- 用於切換到簡單用戶界面的切換按鈕。

提交支持請求

- 您將被引導至Medit Help Center以提交支持請求。

選擇錄像區域

- 用於選擇屏幕的哪個區域錄製視頻。
- 用戶可以記錄程序的整個窗口或僅顯示3D 數據的區域。

視頻錄製

- 開始或停止視頻捕捉。
- 捕獲的視頻文件可以幫助患者、診所和牙科實驗室之間的交流。

截圖

- 用於採集整個屏幕或僅採集掃描軟件的3D 數據顯示區域。
- 捕獲的圖像文件可以幫助患者、診所和牙科實驗室之間的通信。

截圖管理器

- 管理捕獲的圖像。
- 屏幕截圖會自動保存到Medit Link。用戶可以以JPG、JEP、PNG、BMP格式刪除或保存到本地。

Medit i700

+Scan
掃描

修剪

工具

表單資訊

掃描儀狀態

校準

控制掃描深度

復原/重做


工作階段

掃描信息

模型視圖

側面工具欄

掃描階段工具欄

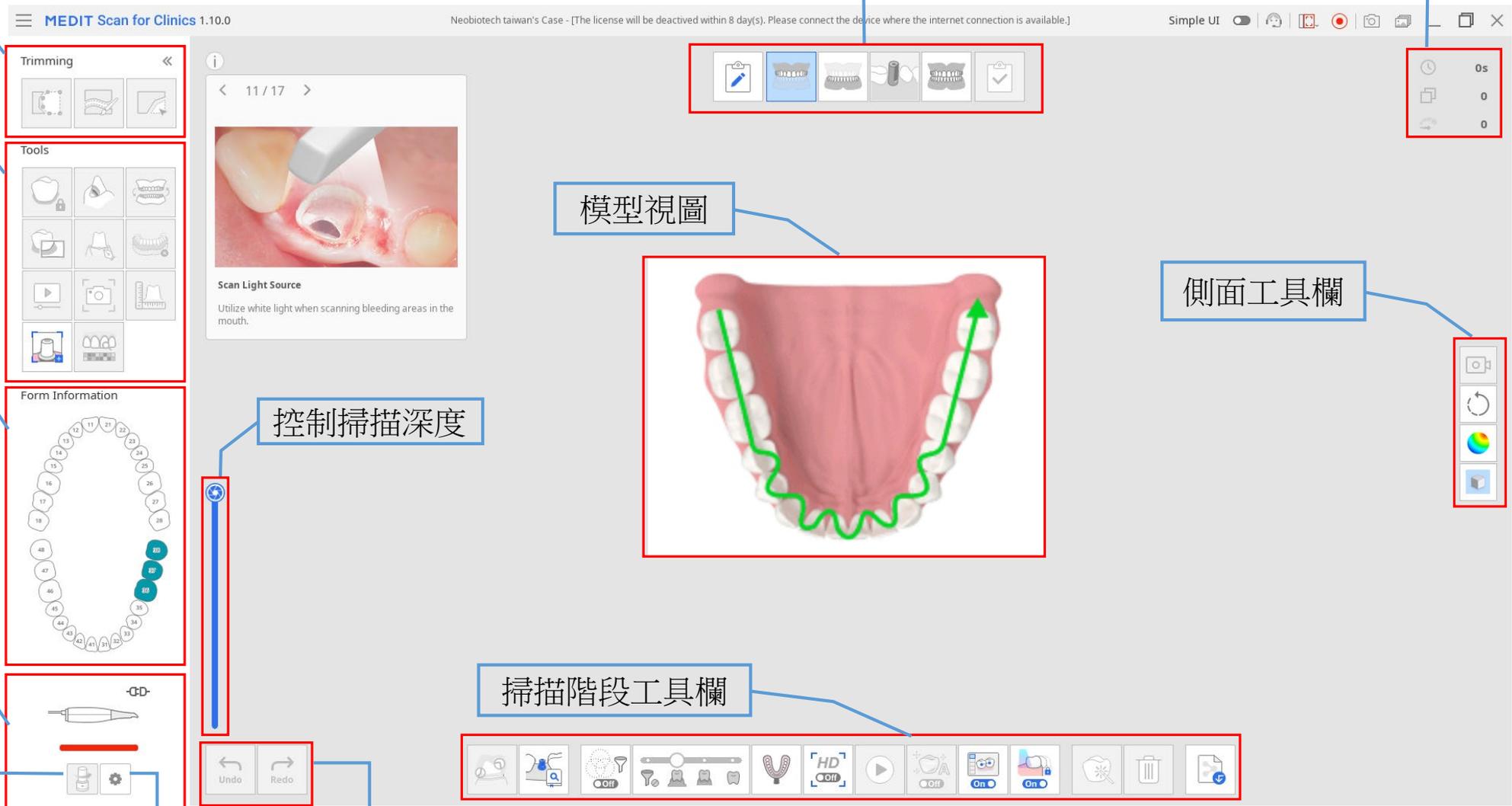

Medit i700

+Scan information

n 掃描信息



掃描時間：顯示掃描期間的掃描時間。



幀數：顯示掃描期間拍攝的圖像數。



掃描速度：顯示當前的掃描速度。

